


OZONE THERAPY ACCESSORIES

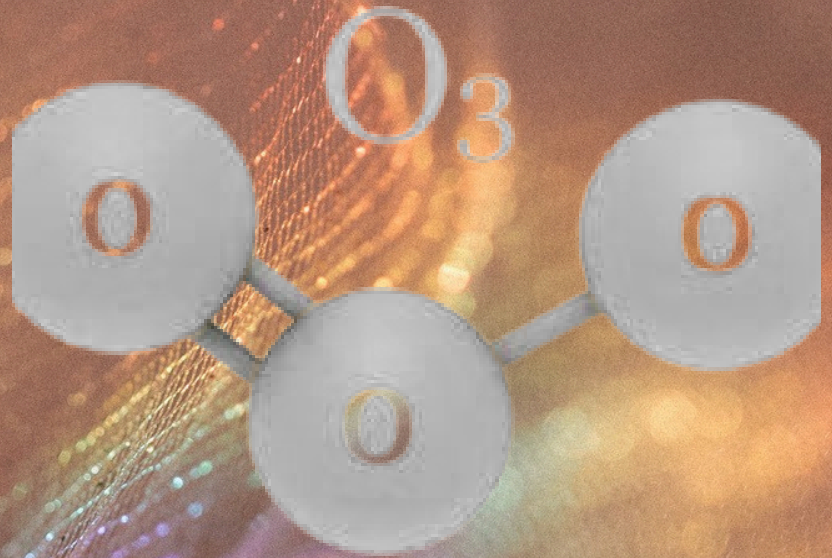

***Not for sale or resale. Only for the use by IAOS trained medical personal only**



Ozone therapy is used to treat a variety of conditions, such as chronic pain and inflammation, arthritis, various bacterial & viral infections, revascularisation of dead or dying tissues, and presently is even being studied as a potential treatment for cancer.

Ozone finds great use in cosmetology, dermatology, shin care, auto immune disorders, and numerous medical dermatological and neurological conditions.

IAOS is the only manufacturer in india to offer medically certified ozone generators with CE marking and is also the only exporter of Ozone machines and accessories to Europe and other developed nations. Here are the various accessories used for Ozone Therapy.



OZONE PROTECTION MASK

THOUGH OZONE FINDS TREMENDOUS USAGE IN MEDICAL PRACTICE, DIRECT INHALATION OF OZONE IS NOT RECOMMENDED. OZONE SMELL IS AN IRRITANT AND ALSO EXACERBATES BRONCHIAL ASTHMA AND COPD.

OZONE INHALATION BY THE TREATING CLINICIAN OR ASSISTANTS IS NOT AN ISSUE IN INTRA ARTICULAR, INTRA DISKAL, PAIN PORTALS, OR MÅHT, WHERE THE OZONE IS DRAWN INTO A SYRINGE AND THE MÅCHINE IS SWITCHED OFF IMMEDIATELY.



HOWEVER IN INSUFFLATION PROCEDURES LIKE VAGINAL, RECTAL, AND OTHER PROCEDURES WHERE OZONE IN AND OUT POINTS ARE THERE, THE EXITING OZONE HAS TO GO INTO A DESTROYER. HOWEVER THERE IS A POSSIBILITY OF LEAK AND THE WHOLE ROOM SMELLING OF OZONE. LIKewise IF THE OPERATION THEATRE IS BEING STERILISED WITH OZONE, FOR THE STAFF TO GO IN AND OUT DURING THE PROCESS INVOLVES BREATHING OF OZONE WHICH IS NOT DESIRABLE.

IAOS OZONE PROTECTION MASKS WITH ACTIVATED CHARCOAL, IS A PATENT PENDING INNOVATION, WHERE AN AIRTIGHT MASK IS USED AND THE INHALATION IS THROUGH A CHAMBER FILLED WITH ACTIVATED CHARCOAL. THIS ASSEMBLY IS HIGHLY RECOMMENDED WITH ALL OZONE PROCEDURES FOR THE THERAPIST AND STAFF.



IAOS Ozone protection masks set of three masks available at a special offer price or Rs 6000/- plus 12% GST.



STAINLESS STEEL OZONE DESTROYERS

Ozone destroyers are needed for all exiting ozone gas so as to not diffuse in air and irritate our pulmonary or nasal mucosa.

Stainless steel is the best ozone compatible material and connecting this to exiting ozone ensures safety for therapist assistant and patient.



IAOS Stainless steel ozone destroyers with activated charcoal and magnesium dioxide, are now available for Rs 12,000/- GST 18% applicable, courier free.



STAINLESS STEEL DIFFUSERS

A therapist needs diffusers for uniform spread of ozone into water for drinking, ozonised water for misting face, ozonised saline for wound washes and pre closure washing, and ozonating oils for cosmetic and therapeutic applications. Putting the other end of a silicone tube into water or oil will bubble ozone no doubt but will not diffuse ozone thoroughly through water, saline or oil. Thus an ozone diffuser is used to uniformly perfuse the liquid with ozone. IAOS machines come with two diffuser stones, but stainless steel diffusers are more convenient with better diffusion capabilities.



**IAOS Stainless steel diffusers at a special offer.
Each diffuser for Rs 4000/- , 2 SS diffusers Rs
7000/- GST 18%**



These diffusers are compatible with IAOS silicone tubes and will last for many many years. Diffusers can be autoclaved, washed and sterilised. It is recommended that the practitioner uses different diffusers for ozonating oils and water. Like wise it is also recommended that the practitioner keeps separate sets for OPD use, one for operation theatre and a couple spares. If the stones become clogged due to repeated use, just boiling them in water and then attaching the open end to a suction machine will clear all clogs and it will start working like new. The pictures below show the role of ozonated oil in regressing leukoplakia and buccal cancer.



CASE OF BUCCAL CANCER IN TOBACCO ADDICT CONTROLLED BY OZONATED OIL APPLICATION



OZONE MAHT KIT WITH SS DIFFUSERS AND LATEX FREE DIFFUSION SETS



***WORLD'S FIRST 200 ML MAHT BOTTLES WITH 96% VACUUM AND SS 316 L DIFFUSER!* 5 BOTTLES, 5 SPECIAL LATEX FREE SPECIALLY MANUFACTURED OZONE COMPATIBLE TRANSFUSION SETS, ONE 50 CC SYRINGE, THREE BIVALVES, SAFE PACKING WITH COURIER RS 8000/- GST 18% FOR BILL.**

***WORLD'S FIRST 200 ML MAHT BOTTLES WITH 96% VACUUM AND SS 316 L DIFFUSER!* 10 BOTTLES, 10 SPECIAL LATEX FREE SPECIALLY MANUFACTURED OZONE COMPATIBLE TRANSFUSION SETS, ONE 50 CC SYRINGE, THREE BIVALVES, SAFE PACKING WITH COURIER RS 15000/- GST 18% EXTRA FOR BILL.**



Major autohaemotherapy is the procedure where 50 ml of patients blood is drawn, and perfused with 50 to 60 ml of 70 micrograms percent ozone. This bubbling diffused ozone makes the blood cherry red. The ozonated blood is now retransfused back to the patient. There are a wide set of indications for MAHT procedure which are described in detail in the book and course lessons.

IAOS MAHT bottles are made out of medical grade flint free borosilicate glasses with stainless steel diffuser pipes to make highest compatibility ozone MAHT bottles in the world. These bottles are equipped with 90% vacuum and hence capable of drawing 50 to 60 ml blood and perfusing it with an additional 50 to 60 ml of 70 micrograms ozone.



500ml MAHT Bottle 5 bottles and sets Rs 20,000 10 bottles and sets Rs 33,000 50 bottles and sets Rs 1.2 lakhs 100 bottles and sets Rs 1.9 lakhs GST APPLICABLE



VAGINAL INSUFFULATION KIT



**VAGINAL INSUFFLATION KIT WITH TWO ONE METRE SILICONE
TUBES INLET AND OUTLET IAOS SPECIAL PRICE FOR RS
12,000/- ONLY. GST 18%**

**Extremely useful for vaginal ozone applications, these
special applicators are made in highly ozone compatible
material, washable, reusable and are an extremely useful
product for women's health.**



6.SILICON TUBES



Silicone tubes: Rs 350/- per metre, minimum 4 metres order.

7.REGULATOR



16000/- plus 18% GST



8.OZONE COMPATIBLE SYRINGES



10 CC latex free syringes



20 CC latex free syringes



50 CC latex free syringes

Ozone compatible syringes (minimum order 50 syringes each) 10 ml Rs 23/- each packet of 50 for Rs 1150/- 20 ml Rs 30/- each packet 50 for Rs 1500/- 50 ml Rs 160/- each packet 50 for Rs 8000/- 150 syringes of all sizes Rs 10650/- plus 18% GST, courier free.




INTRADISKAL OZONE NEEDLE



Intradiskal Ozone Needle Rs 2200/- only. GST 18%

THREE WAY CONNECTOR

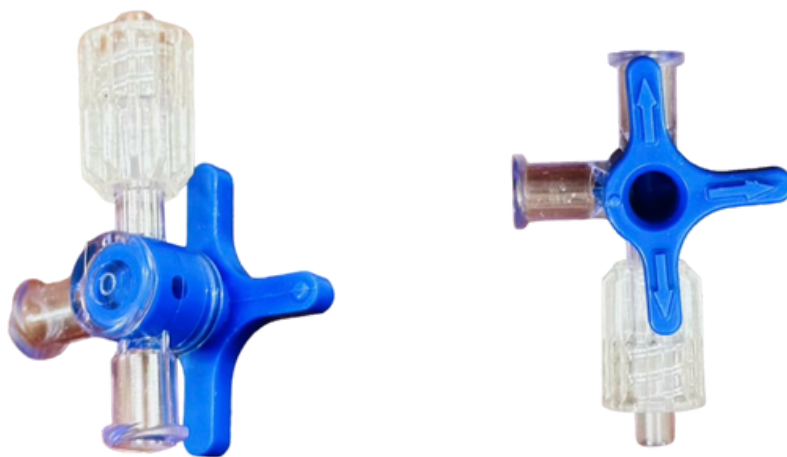

Three way connector Rs 100/- each plus 18% GST



OZONE WOUND BAG



Ozone wound Bag Rs 40 to 75/- each plus 18% GST
(minimum 20 pcs)



OZONE SCALP BAG



Ozone Scalp Bag Rs
200/- each plus 18%
GST (minimum 10
pcs)



MINI OZONE KIT




Dont use for
Blood procedures
MAHT
Infusions
Bladdar wash

Face mist
Ozone oil
Vaginal ozone
Ozone sauna
Drinking water
Vegetable
washing
Wound washing
Ozonated
saline
dressings

**Mini ozone machine kit Air based ozone generator
Silicone tube with diffuser Two Borosilicate bottles with
silicone stoppers Rs 7999/- . 12% GST for bill**



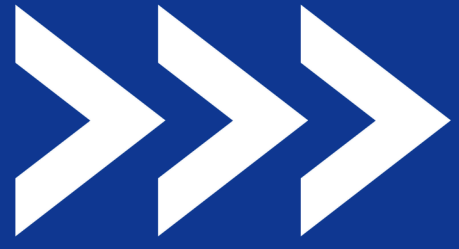

AURAL OZONE STETH



info@iaos.org.in
iaos.org.in@gmail.com



www.iaos.org.in



+91 9791020615



STAINLESS STEEL TUBINGS

SILICONE EAR PLUG

SILICONE TUBINGS

BIVALVE JUNCTION


CONNECTOR

100% ozone compatible






AURAL OZONE STETH

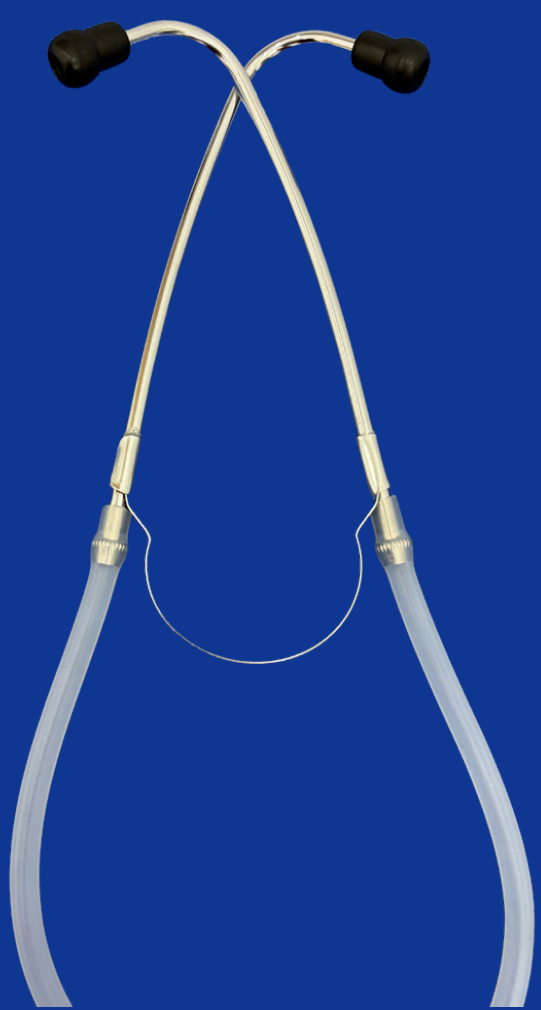


**STAINLESS STEEL
TUBING**

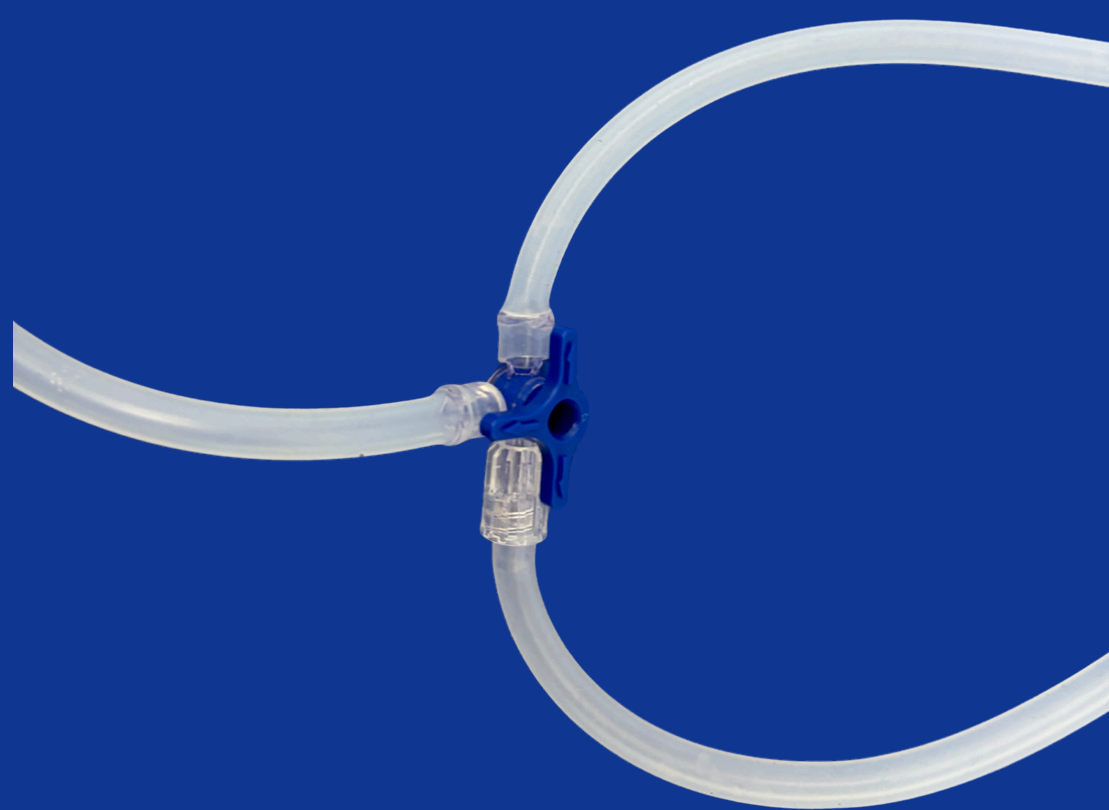

BIVALVE CONNECTOR



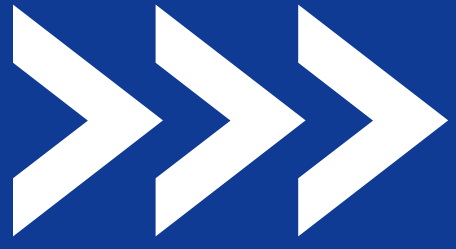

Account Details

Account No:120001617498

Name: IAOS Technology Private Limited

Bank Name : Canara

Bank

IFSC Code: Cnrb0001166

**Rs 8500/-
Special offer
Rs 6000/-**



Share a screenshot of payment with your name and designation to +91 9791020615 WhatsApp. This is very important for you to be included to the WhatsApp group for participants. You will receive the first set of study material in the form of high quality pdf files and lecture link in WhatsApp. Please ensure that you have a valid WhatsApp account to receive the files and links.



**info@iaos.org.in
iaos.org.in@gmail.com**



www.iaos.org.in



+91 9791020615





IAOS

Indian Academy of Orthopaedic Surgeons

Account Details:

Account No:

120001617498

Name: **IAOS Technology
Private Limited**

Bank Name : **Canara
Bank**

IFSC Code:

CNRB0001166

IAOS TECHNOLOGY PRIVATE LIMITED



241922410817408@canb

Share a screenshot of payment with your name and designation to +91 9791020615 WhatsApp. This is very important for you to be included to the WhatsApp group for participants. You will receive the first set of study material in the form of high quality pdf files and lecture link in WhatsApp. Please ensure that you have a valid WhatsApp account to receive the files and links.



info@iaos.org.in
iaos.org.in@gmail.com



www.iaos.org.in



+91 9791020615
(No calls, Whatsapp only)